

COMPLETE RESULTS

C. DIFF QUIK CHEK COMPLETE®

Simultaneously detects GDH and Toxins A and B

- Detects *C. difficile* glutamate dehydrogenase antigen and toxins A and B in a single reaction well
- Yields results in 30 minutes
- Indicates whether *C. difficile* bacteria are present **and** producing toxin
- Provides a clean background and strong signal with patented upflow technology



C. DIFF QUIK CHEK COMPLETE®

The C. DIFF QUIK CHEK COMPLETE® test


Product Order Information

Product Name	Cat. No.	Sample Type	Time to Result	Format
C. DIFF QUIK CHEK COMPLETE®	T30525C/ T30550C	Fresh, frozen, transport media (Cary Blair or C&S)	<30 minutes	Rapid EIA


C. DIFF QUIK CHEK COMPLETE®

Assay Protocol


- 1** Add to a test tube:
- 750 µL *Diluent*
 - 1 drop *Conjugate*
 - 25 µL (1st graduation) of specimen
 - Mix thoroughly

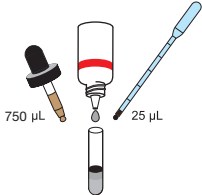


- 2** Transfer 500 µL (highest graduation) from tube to small *Sample Well*. Keep the cassette at room temperature and wait 15 minutes.

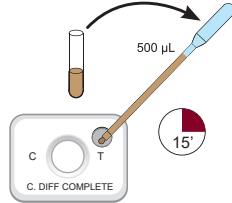

- 3** Add 300 µL *Wash Buffer* to large *Reaction Window*. Allow to completely absorb.

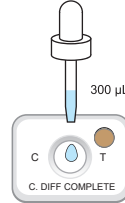


- 4** Add 2 drops *Substrate* to large *Reaction Window*. Keep the cassette at room temperature and wait 10 minutes. Read results.

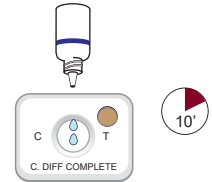


Easy-to-read Results


- Results in 30 minutes without instrumentation
- Uses the popular TECHLAB® QUIK CHEK™ format
- Simple and clear interpretation of results
- Refer to the package insert for additional interpretation of results.

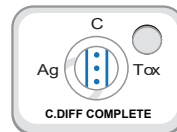

**Positive Ag
Positive Tox**
C. difficile is present
and producing toxins

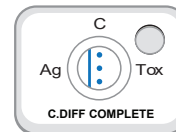


**Positive Ag
Negative Tox**
C. difficile is present

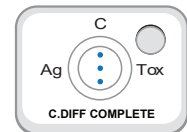

Negative
C. difficile is not
present

C. DIFF QUIK CHEK COMPLETE Package Insert.

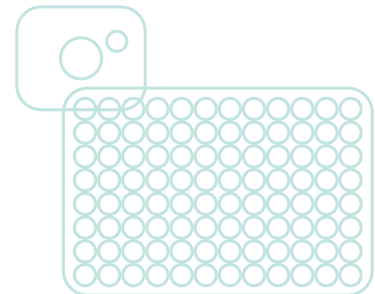

TECHLAB, Inc. is a leading developer and manufacturer of infectious disease diagnostics products worldwide. Since 1989, the company has offered a quality portfolio of diagnostic tests which now includes assays for *C. difficile*, *Campylobacter*, Shiga toxin producing *E. coli*, *H. pylori*, protozoan parasites, and biomarkers of intestinal inflammation.



© 2021 TECHLAB, Inc. All rights reserved.

C. DIFF QUIK CHEK COMPLETE, the TECHLAB Logo, and TECHLAB are trademarks of TECHLAB, Inc. All other trademarks referenced are trademarks of their respective owners. PN 842021185

Distributed by



T 01782 575180 | E enquiries@unahealth.co.uk | W www.unahealth.co.uk

Unit 6&7, Fitzgerald Way, Scotia Road Business Park, Stoke-on-Trent, ST6 4HN

Developed and manufactured by



PROVEN DIAGNOSTIC PERFORMANCE